



Distributed by:
 Wolf Medical Supply, Inc.
 800.335.9653 www.wolfmed.com

GuardIVa^{*}

Antimicrobial Hemostatic IV Dressing



CDC Guidelines recommend the use of a chlorhexidine-impregnated sponge dressing for temporary short-term catheters.

Hemostasis


An *in vivo* animal test showed that, compared to gauze dressings, wounds treated with the GuardlVa* Dressing significantly reduced time to hemostasis and had up to **7 times less blood loss**.^{2,3}



Healing

An *in vivo* animal study showed that the GuardlVa* Dressing allowed wound healing in a manner more consistent with that of untreated wounds as compared to the BioPatch* Dressing.^{2,3}


In Vivo Animal Study Wound Healing after 7 Days

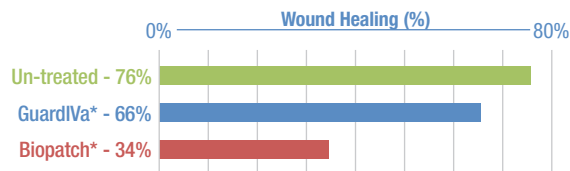

Antimicrobial

Demonstrated *in vitro* antimicrobial efficacy against test organisms known to cause CRBSIs^{2,3}

In vitro testing demonstrated sustained antimicrobial efficacy for up to 7 days. A greater than 4 log reduction in microbial count was observed for all test organisms.

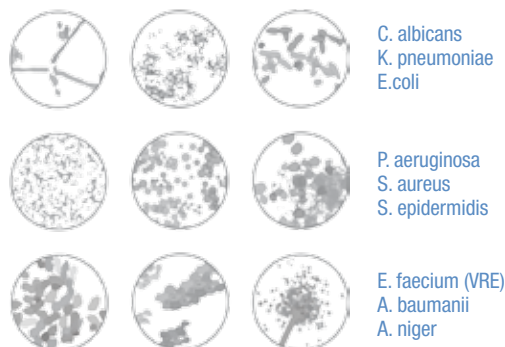

The GuardlVa* dressing has not been clinically tested for its ability to reduce catheter-related blood stream infections.

Code	Disk Diameter	Hole Diameter	Quantity Per Case
FP-27-AD015	0.75" (2.5 cm)	1.5 mm	10/Box • 10 boxes/case
FP-22-AD004	1" (2.5 cm)	1.5 mm	10/Box • 10 boxes/case
FP-23-AD008	1" (2.5 cm)	4.0 mm	10/Box • 10 boxes/case
FP-24-AD012	1" (2.5 cm)	7.0 mm	10/Box • 10 boxes/case



INDICATIONS FOR USE:

The GuardlVa* Antimicrobial Hemostatic IV Dressing is intended for use as a hydrophilic wound dressing to absorb exudate, cover and protect catheter sites. Common applications include IV catheters, other intravenous catheters and percutaneous devices. It is also indicated for control of surface bleeding from percutaneous catheters and vascular access sites.

WARNINGS:

- Do not use the GuardlVa* dressing on patients with a known sensitivity to chlorhexidine gluconate. The use of chlorhexidine gluconate containing products has been reported to cause irritation, sensitization, and generalized allergic reactions. If any such reactions occur, discontinue use of the dressing immediately, and if severe, contact a physician.
- For external use only. Do not allow this product to contact ears, eyes, mouth or mucous membranes.

PRECAUTION:

- GuardlVa* dressing is not intended to treat infection.

³Please consult package insert for more detailed safety information and instructions for use.

Distributed by:
Wolf Medical Supply, Inc.
 800.335.9653 www.wolfmed.com