

WOLF-PAK®



HuberPRO®
Safety Huber Needle Infusion Set

Safety Huber Needle Winged Infusion Set



- **SIMPLE**
- **SAFE**
- **SECURE**

FEATURES

- Non-Coring Needle
- Low profile
- Audible and visible confirmation of safety feature
- Latex free • Non-DEHP
- Easy to use
- Available with or without the needle free access Y-Site

BENEFITS

- Minimizes risk of needlestick injury
- Low profile allows for maximum patient comfort
- Color coded wing for ease of needle gauge identification



CALL: 800.335.9653

— TO ORDER — ONLINE 24/7: www.wolfmed.net

WOLF-PAK®



HuberPRO®

Safety Huber Needle Infusion Set

Needle safety doesn't have to be complicated.



1

Hold as shown and remove Sheath.



2

Insert per protocol.



3

Flip guard forward to lay flat against the patient. Secure per protocol.



4

Push guard forward with index finger to capture the needle as it exits from the patient. An audible "click" will notify you that the needle is captured.

DESCRIPTION

ORDER NUMBER

SHELF BOX QTY/CASE QTY

Micro-bore 19G x 1" w/o y-site

HP1910

25/300

Micro-bore 19G x 1" w/y-site

HP1910Y

25/300

Micro-bore 19G x 3/4" w/o y-site

HP1975

25/300

Micro-bore 19G x 3/4" w/y-site

HP1975Y

25/300

Micro-bore 20G x 1" w/o y-site

HP2010

25/300

Micro-bore 20G x 1" w/y-site

HP2010Y

25/300

Micro-bore 20G x 3/4" w/o y-site

HP2075

25/300

Micro-bore 20G x 3/4" w/y-site

HP2075Y

25/300

Micro-bore 22G x 1" w/o y-site

HP2210

25/300

Micro-bore 22G x 1" w/y-site

HP2210Y

25/300

Micro-bore 22G x 3/4" w/o y-site

HP2275

25/300

Micro-bore 22G x 3/4" w/y-site

HP2275Y

25/300



WOLF MEDICAL SUPPLY, INC.



CALL: 800.335.9653

— TO ORDER — ONLINE 24/7: www.wolfmed.net