



Application of Grip-Lok WA (Wide Adhesive) for PICC Securement


Grip-Lok WA (GL3352) is a catheter stabilization device that is strong enough to lock catheters securely in place, yet versatile enough for almost any securement situation. The adhesive on Grip-Lok WA will adhere and secure catheters made from silicone, PVC and other plastic compounds.

- Soft and flexible fabric design improves patient comfort.
- Provides superior securement for both horizontal and vertical lifting accidental line pulls.
- Simple to apply, inspect and adjust.
- Hypoallergenic, breathable and latex-free to reduce the risk of allergic reactions and skin irritation.

Catheter Securement


Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

1

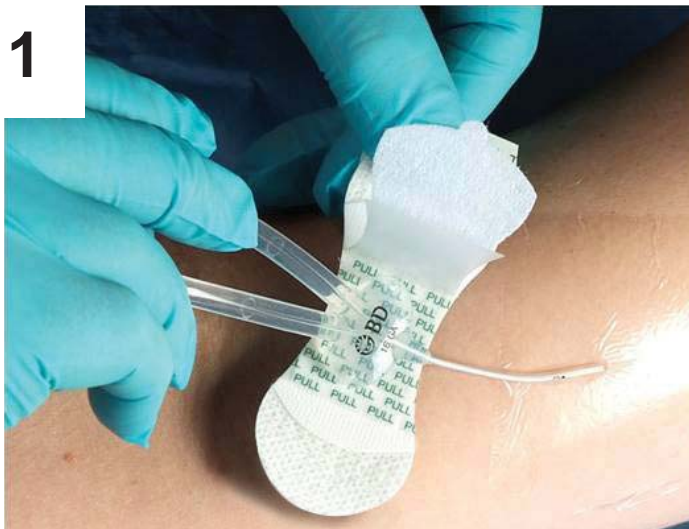

Open the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.

2

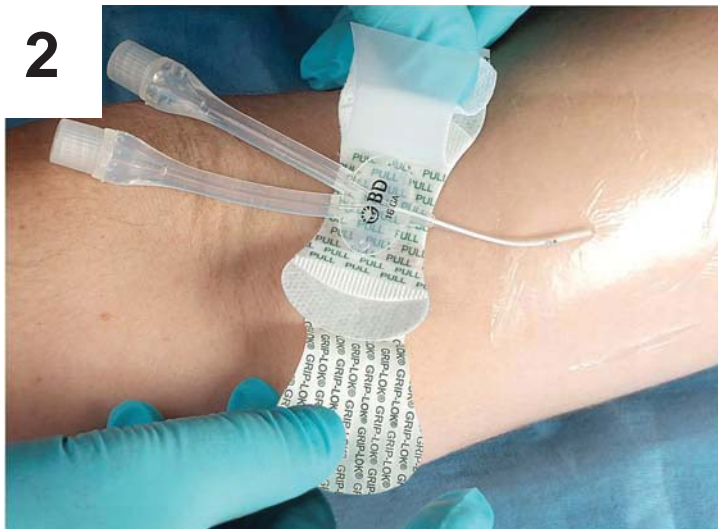

Pull the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.

3



Remove the inside liner labeled PULL to expose the adhesive areas.

4



Place the catheter, tube or line on the bottom adhesive area and secure top flap over the catheter, tube or line. Gently press the top flap down around the edges of the hub to ensure the adhesive connects well to the PICC device and ensure the hook and loop have achieved the best possible connection.

Grip-Lok WA will secure a wide variety of catheter hub sizes and shapes.



Distributed by:



WOLF MEDICAL SUPPLY, INC.

Grip-Lok is a trademark of Zefon International, Inc. Patent#D608887

800.335.9653

www.wolfmed.com